

## Sheep DNA Submission Form

### Instructions:

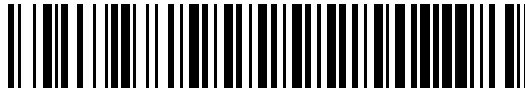
1. Please do not write on this form. Contact the registry for corrections.
2. Thoroughly wash and dry your hands.
3. Sample hair located between or just above hooves. Clean the sample area by removing all loose hair and foreign matter (use comb).
4. Pull (DO NOT CUT) 20-40 hairs strands total. Grasp hair close to skin and pull in one swift downward motion (needle-nose pliers or tweezers can be used, especially if hair is short). Visually check that roots are attached to hair. DO NOT TOUCH ROOT BULBS.
5. Hair sample should be placed in a regular, letter size (4" x 9") envelope (or smaller envelope) and labeled with the animal's name or registration number.
6. Attach hair sample envelope to the bottom of this page as indicated in the shaded box.
7. Place in legal (business size) envelope and mail to the address below.
8. If mailing from outside the United States, a current import permit must accompany the sample.

**Mail To:**

Veterinary Genetics Laboratory - Sheep Testing  
PO Box 1102  
Davis, CA 95617-1102

**For FedEx, UPS, DHL, etc:**

Veterinary Genetics Laboratory - Sheep Testing  
University of California, Davis  
Old Davis Road  
Davis, CA 95616

**Valais Blacknose Intl Sheep Breeders**

Sample ID: VIS\$00000417

**For VGL use**

VIS

Genetic Markers, Parentage

**Owner**

Name: P. Diane Squibb  
Address:

**Animal being tested**

Name: VSR IN06116-2209 Julina  
Registration: 2646 F2  
Color:  
Date of Birth: 05/09/2022      Sex: Female  
Breed: Valais Blacknose  
Microchip:  
Call Name:

**Parent Information****SIRE**

Name: Heartland Aragorn  
Registration: NZ11  
VGL Case #: VIS2

**DAM**

Name: VSR Hershey's Kiss F1  
Registration:  
VGL Case #: SE9804

**ATTACH  
SAMPLE  
ENVELOPE  
(tape or staple)**